



Upgrading to ShowCase 6.5

Overview

SHOWCASE® 6.5 introduces a new level of functionality beyond ShowCase Basic and ShowCase Premier: ShowCase Workstation. ShowCase Workstation has the following key features:

- Query/Retrieve studies from PACS
- Push studies to PACS or Image Center
- Receive studies directly from imaging devices to store in a local (limited) cache.
- Automatically forward studies to PACS or Image Center

ShowCase Premier also has new features in this release:

- Push studies to Image Center in Remote mode.
- Automatic local cache cleanup for automatically downloaded studies when used in Remote mode.

Note: If you need to upgrade from ShowCase Basic with Connect or ShowCase Premier with Connect, please use the *Upgrading from Connect to ShowCase Workstation* guide instead of this one. The process is considerably different.

Doing The Upgrade

The upgrade is straightforward, but it does require that you have Administrator privilege in order to do it.

1. Download the new ShowCase DICOM Viewer installer from here: <https://www.triltech.com/download.html>
2. Uninstall your existing ShowCase 6.1.x software. All of your data, configuration, and license settings will still be on the computer.
3. Run the installer you downloaded in step 1.
4. Start ShowCase

Subtle and Not-So-Subtle Changes in ShowCase

- The Study List is no longer constrained to live inside the main ShowCase window. It is a separate window, which can be configured to always remain on the screen or close when a study is open. There are new options in the Edit => ShowCase Settings... to control this.
- The File menu has been simplified, to reduce clutter.
 - o Create/Repair DICOMDIR functions are now located in the button bar at the bottom of DICOMDIR folder tabs in the study list.
 - o License Management is now under the Help menu.
- The use of the DICOM-centric word “Series” has been replaced with “Study”. This reflects an overall change in behaviors and user interface to be more study centric, to reflect the needs of our customers.
- If you want to view raw DICOM image information, click on an image and go to the Study menu and select View Dicom Image Tags.

SHOWCASE® is a registered trademark of Trillium Technology, Inc.